

NeuroTherm®

Always
a step
ahead...

Disposable StimJect®

No Removing Your Electrode


The best way to inject local without having to remove the RF electrode

No Losing Your Threshold

Not having to remove and reinsert the electrode means the cannula stays at the exact location of your stimulation threshold

No More Lost Time

StimJect cannula saves you time allowing you to move from patient to patient more quickly

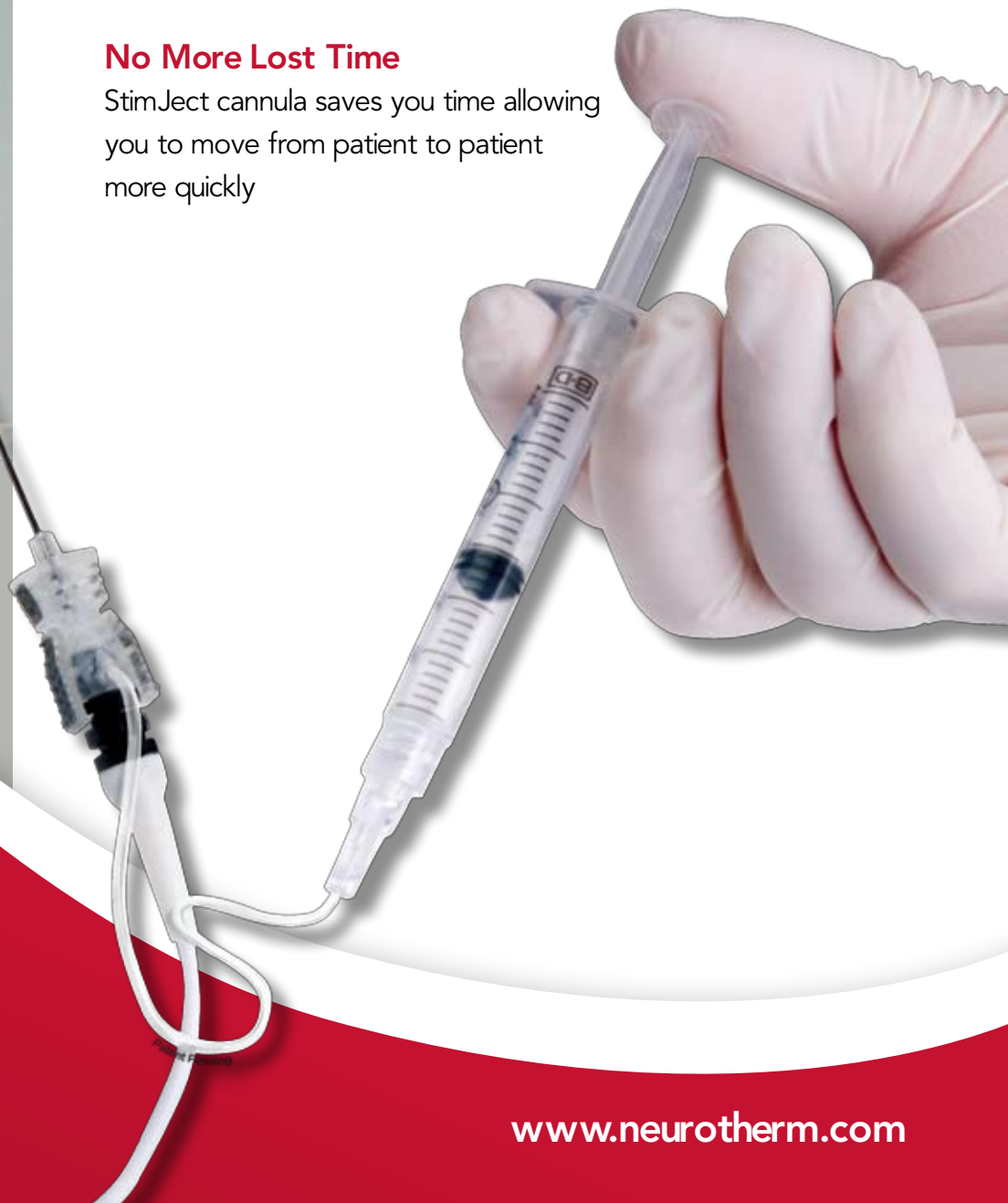

NeuroTherm®

30 Upton Drive, Suite 2
Wilmington, MA 01887
USA

Tel: +1 888 655 3500

Fax: +1 978 658 2378



NeuroTherm®

429 Brighton Road, Croydon
Surrey, CR2 6EU
United Kingdom

Tel: +44 20 8660 4374

Fax: +44 20 8660 9417

NeuroTherm®

Modemstraat 20
1033 RW Amsterdam
The Netherlands

Tel: +31 204 702 702

Fax: +31 204 702 701

www.neurotherm.com


The New Disposable StimJect®

Our new disposable StimJect is designed for use with NeuroTherm radiofrequency RF electrodes in (P)RF procedures. Place the StimJect cannula as you would a regular cannula. Remove the stylet and insert the electrode, then follow the usual stimulation procedure with the RF generator.

Once the desired location and threshold have been achieved, insert local anaesthetic into the injection tube rather than having to remove the electrode. The electrode movement is minimized as it is locked firmly into place by the StimJect cannula. Proceed to lesion as usual.



The StimJect cannula can only be used with a NeuroTherm RF Generators and Electrodes.

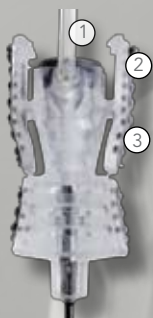

- 1 Side injectionport
- 2 Clasp firmly secures electrode
- 3 Moulded grip for enhanced control

5 cm Curved Sharp				
Length	Description	Tip	Gauge	Part #
5 cm	Straight StimJect	5 mm	20	SJ-S-505-20
10 cm	Straight StimJect	5 mm	20	SJ-S-1005-20
10 cm	Straight StimJect	10 mm	18	SJ-S-1010-18
10 cm	Straight StimJect	10 mm	20	SJ-S-1010-20
15 cm	Straight StimJect	10 mm	18	SJ-S-1510-18
15 cm	Straight StimJect	5 mm	20	SJ-S-1505-20
15 cm	Straight StimJect	10mm	20	SJ-S-1510-20
5 cm	Curved StimJect	5 mm	18	SJ-C-05-18
5 cm	Curved StimJect	5 mm	20	SJ-C-505-20
10 cm	Curved StimJect	5 mm	20	SJ-C-1005-20
10 cm	Curved StimJect	10 mm	18	SJ-C-1010-18
10 cm	Curved StimJect	10 mm	20	SJ-C-1010-20
15 cm	Curved StimJect	10 mm	18	SJ-C-1510-18
15 cm	Curved StimJect	10 mm	20	SJ-C-1510-20

"The StimJect® allows me to inject without the need to remove my RF electrode. Without the worry of needle displacement during injection, I am confident of correct patient treatment. This is quite an innovative breakthrough in RF cannula technology!"

Dr. Richard Rosenthal, Medical Director of Nexus Pain Care, Provo, Utah, USA

Distributed by:

www.neurotherm.com



ISO 13485 Certified

NeuroTherm is a registered trademark of NeuroTherm Inc.